




Holter Hook-Up Chart **BOXER**

Our Holter model is Ch. Encore's Stetson

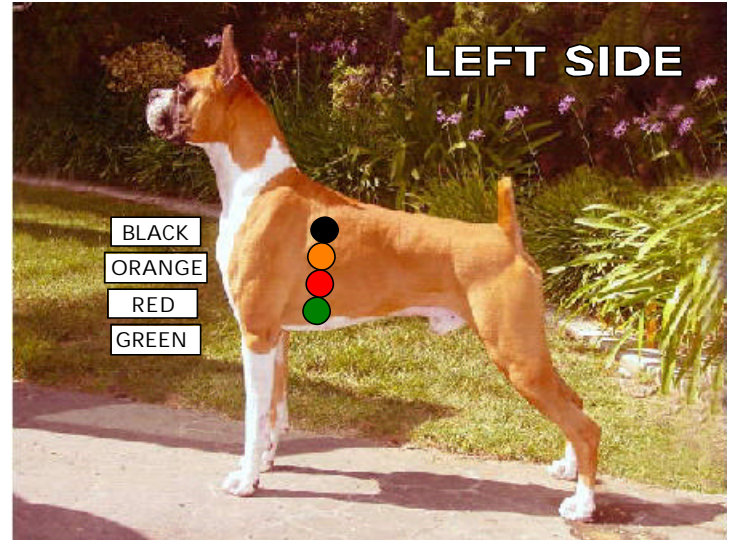

PLEASE NOTE: Not all machines have the Blue and Orange leads. If you have one of these 5-wire machines, skip the blue and orange and move the black and white down one step to be in line with the others.

ANIMAL PREPARATION

Have the dog stand normally. If the dog can be elevated on a table it will be easier. Shave both sides of the chest. The shaved area should be located behind the elbow and should be large enough to accommodate the electrodes. Rub the shaved areas with alcohol to remove debris and oil. Allow enough time to dry completely. A vest is recommended to protect the wires and electrodes. If you don't have a vest, tape over wires with elastic tape (Expandover®). Make sure not to stretch the elastic tape excessively or the dog will be uncomfortable.

OPERATING THE HOLTER MONITOR

1. Insert Cassette.
2. Install new battery. Polarity (+/-) doesn't matter.
3. Connect Animal.
4. Close Head Bar. Make it "click" twice.
5. Visually check that the tape is turning.
6. Write animal name, start time and date on cassette.
7. Complete Holter Monitoring Request Form.
8. When finished monitoring, send the cassette, the form and the diary to Alba Medical for scanning.

ALBA MEDICAL SYSTEMS, INC.
Administrative Office 877-482-1001 Scanning Office 866-ALBA-MED
More information at www.albamedical.com