




## Holter Hook-Up Chart **CANINE DR180 Holter**

Our Holter model is Ch. Encore's Stetson

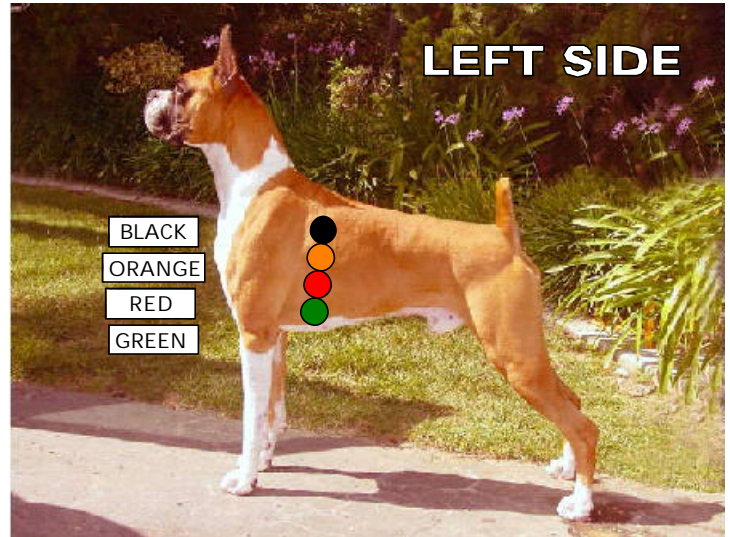

### ANIMAL PREPARATION

Have the dog stand normally. If the dog can be elevated on a table it will be easier. Shave both sides of the chest. The shaved area should be located behind the elbow and should be large enough to accommodate the electrodes. Rub the shaved areas with alcohol to remove debris and oil. Allow enough time to dry completely. When fully connected, tape over wires with elastic tape (Expandover®). Make sure not to stretch the elastic tape excessively or the bandage will be too tight and the dog will be uncomfortable.

### OPERATING THE HOLTER MONITOR

1. Insert compact flash card into the recorder.
2. Insert 2 new AA batteries. Press NEXT.
3. Press NEXT or NEW PATIENT.
4. Enter patient ID. (You must enter some number - up to 20 digits).
5. Press NEXT.
6. Lead quality or impedance should be 3, 4 or 5. Press NEXT.
7. Press NEXT to cycle through the channels.
8. Press START to begin recording.